



Compatible with any
2, 4 or 6 Point
Spreader Bar

PATIENT SPECIFIC SLINGS SIT 2 STAND

Can Be Used For Bathing & Showers, Pools

With Key Hole Opening Clips



Also available v
"Key-Hole"
Opening Clips

GENERAL INFORMATION

Your Patient Specific (Disposable) Sling is a High Quality Product with many safety features incorporated within the material design and content of the product. The Slings are available in
Slings with Straps - 3 Sizes of Small, Medium and Large
Slings with Key Hole Opening Clips - 3 Sizes of Small, Medium and Large

Each Sling is size/colour co-ordinated on the Binding, Handles, Displacement Straps.

The Slings are approved for a load of 300 Kg (660 lbs) or 500 Kg (1100 lbs) for the Extra Extra Large Size Sling. The Patient Specific Slings are compatible with most hoists (lifters) including hoists (lifters) with Key Hole Opening Clip Spreader Bars.

The Patient Specific Slings can also be used for Bathing and Showers and also in Pools. Use with confidence - the Slings are not going to rip or tear or fall apart! Dispose of sling after bathing, etc and replace with new sling.

The Patient Specific Slings can also be left behind under the Patient. The Slings are 100% breathable. The Patient Specific Slings are intended to be used as a Patient Specific Product and should not be recycled and re-distributed.

Dispose of Sling after use.

SLING COMPATIBILITY

The Slings with Webbing Attachments are compatible for use with any 2 Point Spreader Bar Lifter System.

The Slings with Key-Hole Opening Clip Attachments (Buckles) are compatible for use with Lifter Systems where such systems provide for Key-Hole Opening Buckle-Lock Spreader Bar Attachments.

Wash Indicator

The Patient Specific Slings incorporate a Wash Indicator Label. In the event of the sling being washed (and then recycled for use again) the Wash Indicator will have changed design so that the Care Provider is aware that the Sling has been washed.

Do not re-use and dispose of the Sling.

Prior to Washing

WASH




The Wash Indicator will highlight that the Sling has been washed. Do Not Use the Sling!



After Washing

WASH

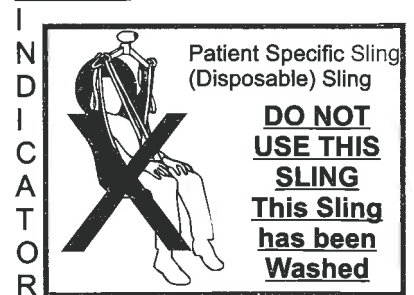


FOR MORE INFORMATION OR INQUIRIES PLEASE CONTACT YOUR LOCAL REPRESENTATIVE
OR PRACTICAL PRODUCTS FOR LIVING INC.


Practical Products For Living Inc.
1914 Winding View, San Antonio, TX 78260

SIT 2 STAND FITTING INSTRUCTIONS

SIT 2 STAND SLINGS - FITTING THE SLING

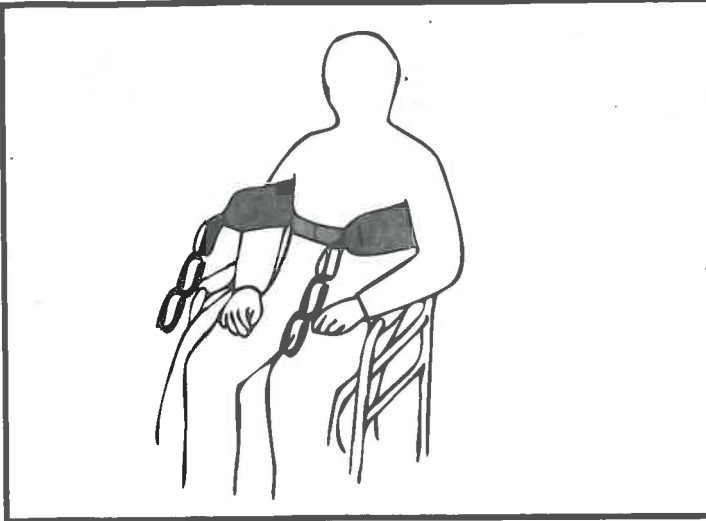

Place sling down behind the patient's back between base of shoulder blades and bottom of rib cage. Attach velcro strap around the front of the Patient. It is there to hold the sling in position whilst attaching the sling to the Sit 2 Stand Lifter. Ensure that the patient's arms are outside - as indicated above.

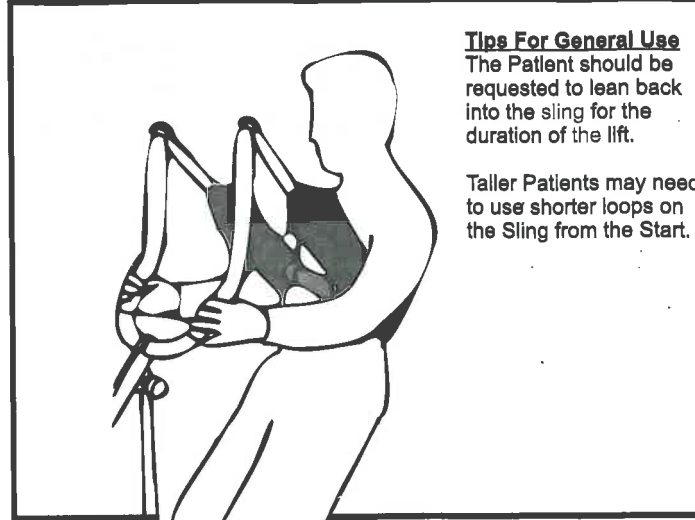


Tips For General Use
The Patient should be requested to lean back into the sling for the duration of the lift.

Taller Patients may need to use shorter loops on the Sling from the Start.


Position the Sit 2 Stand Lifter so that the patient's feet fit in the foot trays and knees against the knee pad. Initially attach the longer loops of the sling to the hooks of the Lifter. Ensure the patient grips on the hand grips supports. Once the patient is used to the lift, the shorter loops on the webbing straps may be used for a more upright position. The patient is now ready to be raised.

SIT 2 STAND - ALTERNATIVE ATTACHMENT (FITTING) OF THE STRAPS

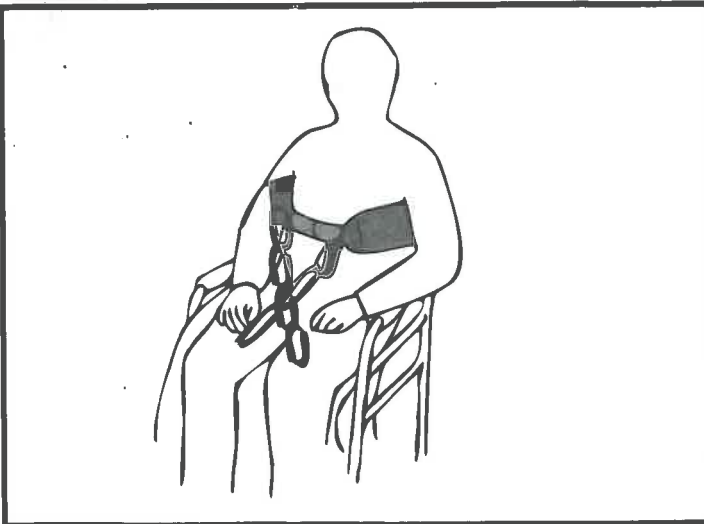

Criss-Cross the Hoist (Lifter) Attachment Straps - take one of the straps and thread through the other - as in the diagram above.

You can criss-cross through any of the loops.

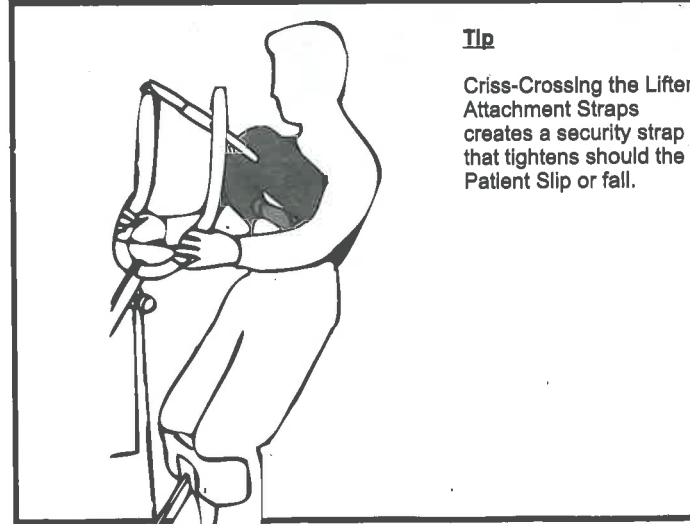

Tip

Criss-Crossing the Lifter Attachment Straps creates a security strap that tightens should the Patient Slip or fall.