
Safety recommendations

- Read the Instructions for use carefully (find this at: www.lopital.com) before use.
- Maximum load 220 kg.
- Make sure that the chair is on the brake during client transfers from and to the device.
- Place both feet on the foot supports.
- Armrests are closed during up / down movements and arms should rest on armrest.
- Only relocate and move the chair in the lower position.

Operation panel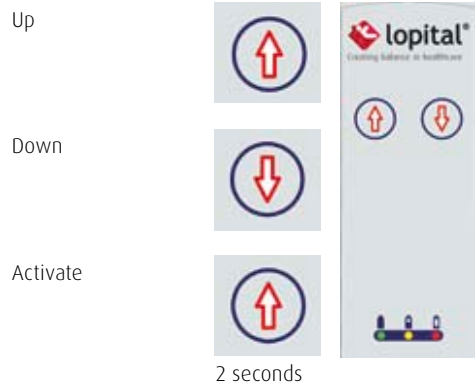

Important!

This instruction card may not be considered a replacement for the Instructions for use.

Use

1. Activate the Elexo XXL by pressing the 'up' button for 2 seconds.
Note: After use, the Elexo XXL will go into a sleep setting after 60 minutes.
2. Place the chair in the right position and fold the foot supports up.
3. Apply the brake to the chair!
4. Fold the armrests up if necessary.
5. Support the less able person when they are taking a seat.
6. Shut the armrests and let the client's feet rest on the foot plates.

Stand up by carrying out the same procedure in reverse order.

Wheel blocking

Foot rest

Both foot plates must be turned away for the less able person to take a seat in the chair. Make sure that the person places both feet on the foot plates when the chair is moving.

Armrest

1. Unlocking.
 2. Turn backwards.
- Close by repeating process in reverse order.

Charging batteries

After using the Elexo XXL, connect it to the charger to maintain the batteries.

- | | |
|-----------------|---------------------------|
| Off | Sleep setting. |
| Green | Chair ready to use. |
| Blinking orange | Emergency stop activated. |
| Orange | Battery is half empty. |
| Blinking red | Battery almost empty. |
| Red | Battery empty. |


The Elexo XXL will not work during charging.

Toilet function

- *In combination with toilet:* The Elexo XXL can be moved over almost every toilet. Move the chair upwards or downwards to achieve the right position.
- *With bedpan:* The Elexo XXL can also be used as a toilet chair. Simply slide the stainless steel / plastic bedpan under the chair using the guides.

Emergency lowering

If the Elexo XXL breaks down, you can lower the chair using the emergency lowering button.

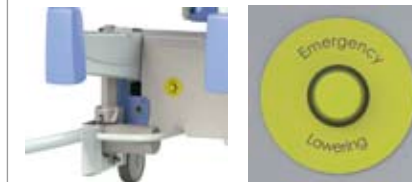Cleaning and maintenance

The Elexo XXL must be cleaned after every use, with water and regular household cleaning agents. Do not use any abrasive or other aggressive cleaning products. Check the wheels regularly for hairs, dust and soap residues and remove when necessary.

The complete maintenance schedule can be found in the Instructions for use (**find this at: www.lopital.com**).

In the event of a breakdown, contact Lopital.
Tel: +31 (0)13 - 5239300, **Fax:** +31 (0)13 - 5239301
E-mail: service@lopital.nl

Instructions for use

For the full Instructions for use go to **www.lopital.com**

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