



Lopital Elexo XXL Skills Check Off List

Competency Statement: After attending this orientation, the caregiver will be able to properly use the Lopital Elexo XXL Shower Chair. If the caregiver has any questions, always refer to the product manual prior to use, or request additional training.

Safety Knowledge	Acknowledged & Understood
Weight capacity of 485 lbs.	
Brake on during transfers	
Foot rest are to be at right position	
Feet are to be on foot rest	
Head support to be set at correct position, when used	
Front arms in down position during up/down	
Arms should rest on armrest	
Green - Chair ready for use	
Blinking Orange – Emergency stop activated	
Orange – Battery is half empty	
Blinking red - Battery almost empty	
Red - Battery dead	
Elexo XXL will not work while charging	
Elexo goes into sleep setting after 60 minutes of non-use	
Lid on seat to be removed for toileting	
Patient/Resident transport of chair in lowest position only	
Cleaning and Maintenance	
Must be cleaned after each use	
Quaternary/household cleaners acceptable	
No Phenol based cleaners	
Check wheels regularly for debris	
Operation Use	Completed
Locate and press up button for 2 seconds to activate	
Locate/use up button	
Locate/use down button	
General Use	
Apply/release brakes on each wheel	
Unlock and raise armrests	

Lower and lock armrests	
Insert and Remove Bedpan	
Remove footrest	
Place footrest back into proper position	
Secure footrest	
Charging/Battery	
Plug magnetic charger to lift	
Unplug magnetic charger from lift	
Toileting function	
Move Elexo XXL over a toilet	
Slide commode pan under Elexo seat	
Emergency lowering	
Locate emergency lowering button	
Depress button to lower chair	

Facility name _____

Staff Member _____

Date _____

Supervisor _____

In-service by _____

Trainer company _____