



## Lopital Flexo Skills Check Off List

Competency Statement: After attending this orientation, the caregiver will be able to properly use the Lopital Flexo Shower Chair. If the caregiver has any questions, always refer to the product manual prior to use, or request additional training.

<b>Safety Knowledge</b>	<b>Acknowledged &amp; Understood</b>
Weight capacity of 330 lbs.	
Brake on during transfers	
Foot rest are to be at right position	
Feet are to be on foot rest	
Head support to be set at correct position, when used	
Front arms closed during up/down if used	
Arms should rest on armrest	
Lid on seat to be removed for toileting	
Movement of chair in lowest position only	
<b>Cleaning and Maintenance</b>	
Must be cleaned after each use	
Quaternary/household cleaners acceptable	
No Phenol based cleaners	
Check wheels regularly for debris	
<b>Operation Use</b>	<b>Completed</b>
Locate foot pedal for up/down feature	
Move chair up by pumping foot pedal	
Move chair down by depressing foot pedal and holding it down	
<b>General Use</b>	
Apply/release brakes on each wheel	
Unlock, open and raise armrests	
Lower and lock armrests	
Insert and Remove Bedpan	
Remove footrest	
Place footrest back into proper position	
Hook and lock footrest	
If straight arm rests, move up and down	

## Toileting function

Move Flexo over a toilet	
Slide commode pan under Flexo seat	

Facility name \_\_\_\_\_

Staff Member \_\_\_\_\_

Date \_\_\_\_\_

Supervisor \_\_\_\_\_

In-service by \_\_\_\_\_

Trainer company \_\_\_\_\_