Lopital Marina PM Checklist

Marina Part #:	
Serial Number:	
Date:	


1. General condition of the trolley

Month	12	24	36	48	60	72	84	96	108	120
	С	С	С	С	С	С	С	С	С	С

- · Visually check undercarriage, levers, castor caps, side barriers and push handle for any damage, and touch up any painted parts where necessary.
- · Replace insert caps if they are damaged.
- CE sticker should be present and clearly legible; if this is not the case it should be replaced. Also check the sticker with the inspection date.
- In the case of widespread corrosion, judge whether this is affecting the structure's strength.

2. Side barriers

Month	12	24	36	48	60	72	84	96	108	120
	S	S	S	S	S	S	S	S	S	S

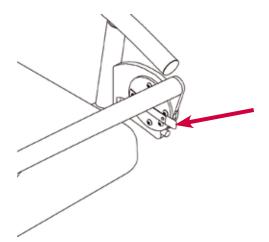

- Both side barriers should open and close smoothly.

- The black or red submersible cap should be undamaged.____

· Lubricate the hinge between the two hinge plates (preferably with Innotec Ceramic Grease, #03.0121) after removing any • They should be easy to lock and unlock. excess dirt and corrosion. __ · Check whether all the bolts are present and tightly screwed.

Notes on 1 and 2:

Required tools

• Touch-up pen Sikkens 477A2.

Lubricate

- · Digital spirit level.
- · Marina maintenance support
- Allen keys (3, 5, 6, 8 mm).
- Open-end/ring spanners (10, 13, 17, 19 mm).
- · Ceramic grease spray 03.0121.

3. Castors + brakes


Month	12	24	36	48	60	72	84	96	108	120
	С	С	С	А	С	С	С	А	С	С

Required tools

- · Allen key 8 mm (shortened).
- Open-end/ring spanner 17 mm.
- Lock nut M10 (4x).
- Derrick for Marina trolley.

- All four the castors should touch the floor on a flat surface.
- If the castors are damaged or worn they have to be replaced; for instance when parts of the tread are missing or when the brakes are no longer working or work very badly.
- If the castors are very stiff, you should try to solve this by cleaning them. If this means they have to be detached, they should be refitted afterwards with a new lock nut.

It is recommended to replace the castors once every 4 years.

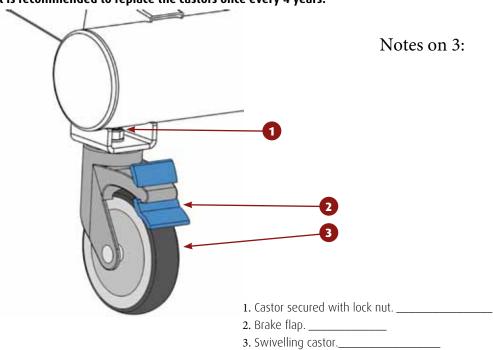

4. Mattress + pillow

Check for any damage and replace where necessary.	
	Notes on 4

Failure items:

- Damage. _____
- Torn fabric.______
- Moisture in mattress._____
- Severe discoloration. ______
- Severe mold (mainly on underside). Clear______ Needs Cleaned______

Have the mattress to be cleaned thoroughly and check again. (IF REQUIRED)______

For reasons of hygiene, it is recommended to replace mattress and pillow once every 5 years.

5. Outlet hose

Month	12	24	36	48	60	72	84	96	108	120
	С	С	А	С	С	А	С	С	А	С

Required tools

• -

Check for any kinks, leaks or blockages._____

Failure items:

- Kinks. _____
- .

.

Blockage. ______

· Cracked swivel._____


Outlet damaged.______

For reasons of wear and tear, blockage and hygiene, it is strongly recommended to replace the outlet hose once every 3 years.

Notes on 5:

1.	Outlet.
2.	Seal.

3. Outlet hose.

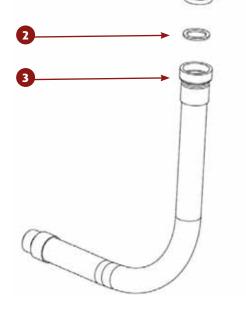

6. Operation, emergency stop and emergency lowering device (61002350 electric)

Required tools


· Multimeter.

- 1. Connection for manual operation.
- 2. Control box.
- 3. Emergency stop.
- 4. Emergency lowering button.

Month	12	24	36	48	60	72	84	96	108	120
Emergency stop	C	С	С	А	С	С	С	А	С	С
Emergency	С	С	С	С	С	С	С	С	С	С
lowering device										
Operation	C	С	С	С	С	С	С	С	С	С

· Check all functions (also pay attention to any noise).

- When the emergency lowering button is pressed, the lying surface should lower. _____
- When the emergency stop is pressed, the trolley should not respond to any command.
- When the electric connections are corroded, they have to be cleaned and sprayed with contact spray. _____
- Replace manual operation if the cable is damaged, kinked or overstretched.
- Check cable of the actuator for any kinks or other damage.____

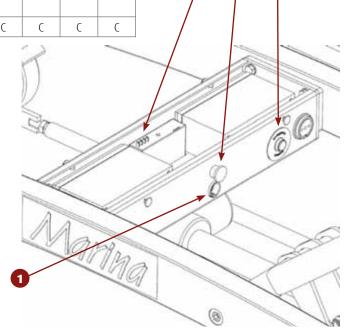

7. Battery + charger (61002350 electric)

Month	12	24	36	48	60	72	84	96	108	120
	С	С	A	С	С	А	С	C	А	С

Required tools

- · Tiewraps.
- · Cutting pliers.
- · Multimeter.
- · Contact spray.


- · A battery can be checked only when it is charged! ___
- Measure batteries separately and replace if < 4.5A (< 65%). Always replace both batteries _

· Write the replacement date on the battery (waterproof pen) and put a date sticker on the rear of the motor

1. Battery (2x).

2. Charger connector.

- · Check charger for any damage (plug + cable + magnet contact); if the cable is damaged, replace the whole charger.
- · Check the charging voltage; it should be at least the value as indicated on the charger, and be more than 27 Volt.
- · Check whether there is too much corrosion on the connectors; also check the charger connector on the trolley. Unplug the mains plug and spray it with contact spray.

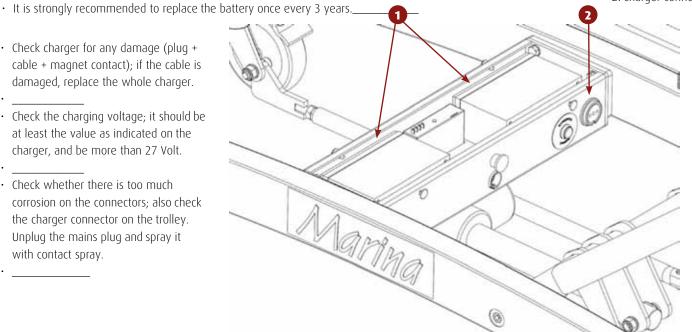

8. Hefboom


Month	12	24	36	48	60	72	84	96	108	120
	С	RC	С	RC	С	RC	С	RC	С	RC

- · Check for any damage. If the lever is bent, it should be replaced. If it is from aluminium, it should be replaced as a precaution. _____
- Remove insert cap.
- If the bearings of the levers have been worn away to an extreme level, the trolley should be returned to Lopital._
- · Check retaining rings (8 in total; they connect the subframe to the two levers). _

- 1. Insert cap.
- 2. Lever.
- 3. Bearing.

Required tools

- · Soft hammer.
- · Flat screwdriver.
- · Digital spirit level.
- · Retaining ring pliers.

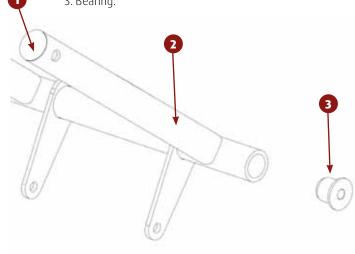

9. Stootrollen

Month	12	24	36	48	60	72	84	96	108	120
	С	С	А	С	С	А	С	С	А	С

Tech Signature:

Required tools

- Open-end/ring spanner 10 mm.
- · Loctite (blue).
- · Ceramic grease 03.0120.

The buffer discs should rotate smoothly		
If the buffer discs are damaged or worn, e.g. parts of the missing, they should be replaced.	ne disc	
 It is recommended to replace the buffer discs once eve 	ry 4 years.	
 Method Unscrew bolts Inspect sleeves and buffer discs (if damaged: new ones). 	 Fitting sleeve/cap. Buffer disc. M6 bolt. 	
• Put grease on the sleeves	2	
 Put Loctite on the thread of the M6 bolt Tighten buffer discs 		
Date/Time PM Completed		_
Marina Part Number		_
Marina Serial Number:		_