

# ProCare Medical

healthcare solutions <sup>2</sup>

Easy 6 step instructions to attach your lapbelt to the your Tango Shower/Commode Chair- The only tool required is a phillips head driver.



1) Locate space between frame members and seat as shown in picture 1.



2) Locate seat screw on right rear bottom side of the seat.




3) Remove seat screw as shown in picture 3. (Requires phillips head driver).




4) Verify that you have belt assembly and two washers before proceeding.



5) Place washer on screw, then screw through pre-punched hole on one of the ends of the belt assembly. Washer must be applied as shown for each assembly.

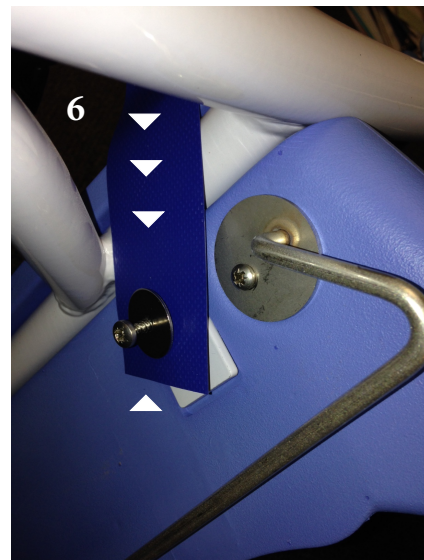

6) Run strap and screw through the frame arch identified in step one, then screw the completed assembly back onto seat bracket.

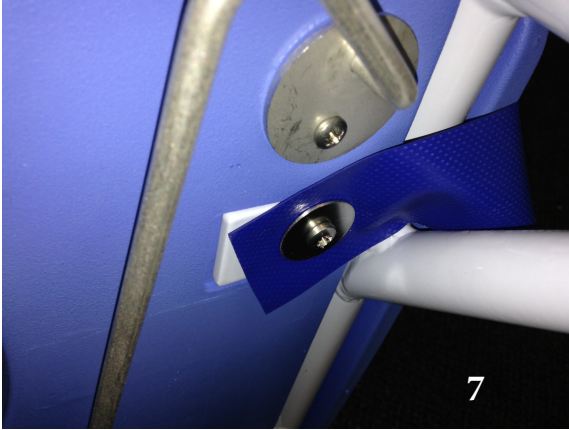

7) Repeat steps 1 through 6 for the opposite side of the chair.



8) The completed attachment of the lap belt assembly should look like this from the bottom of the seat

9) The completed assembly should look like this from the top of the seat.



*If you have any questions or have any difficulties attaching your lap belt to your Tango Shower/ Commode chair please contact ProCare Medical's Service Department at 855-528-0421*