

Tango XXL PM Checklist

Tango XL Serial Number: _____

1. General condition of the chair

Date: _____

Month	12	24	36	48	60	72	84	96	108	120
	C	C	C	C	A	C	C	C	C	A

Required tools

- Touch- up pen Sikkens A77A2

- Visually check if the log, the instructioncart and the user manual are available. When it's not available, report this by the user. _____
- Visually check if all stickers are present and legible. Can you read the serial number yet? _____
- The CE Sticker should be present and clearly legible. This also applies to the sticker with inspection date. _____
- Visually check all painted parts for damage and corrosion. Touch up if necessary. _____
- Check PUR parts for severe discolouration (Brown in particular) and damage. _____
- Are all the screws present and securely fitted? Also that of the guide? _____

Present:

Screws Present Y ___ N ___ Screws replaced? Y ___ N ___

- Foot supports (not a failure item) **Present** :Y _____ N _____

For reasons of hygiene, it is strongly recommended to replace the seat and back once every 5 years.

2. Wheels

Month	12	24	36	48	60	72	84	96	108	120
	C	C	C	C	C	C	C	C	C	C

Required tools

- Allen key 8mm



Figure 1 Wheel

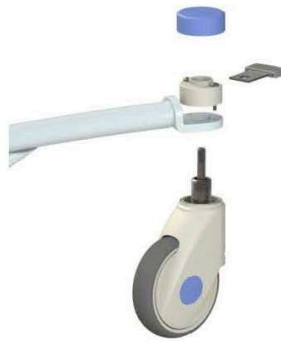


Figure 2 wheel exploded

- Check the attachment of the wheel (Tolerance) _____
- Check freewheel function + brakes. _____
- Do all four the wheels touch the floor on a flat surface? If not, check the chair right course. _____
- Brake lip intact? _____
- Check the tread. _____

Replace the wheels if they are damaged or worn.

Wheels Replaced? Y ___ N _____

3. Arm rests

Month	12	24	36	48	60	72	84	96	108	120
	C	V	C	V	C	V	C	V	C	V

Required tools

- Allen key 6mm

- Inspect its functioning, and check for any damage, corrosion. _____
 - Check its room for motion, measured at the front point (replace if it is more than 4 cm.) _____
 - Always treat corrosion.
 - Are the arm supports evenly positioned? **Y**_____ **N**_____
 - Is the secure bolt present? **Y**_____ **N**_____
- REPLACE IF MISSING** _____



Figure 3 Armrest

Replace bolts once every 2 years.

Look for an exploded view on www.lopital.nl

4. Foot support

Month	12	24	36	48	60	72	84	96	108	120
	C	C	C	C	C	C	C	C	C	C

Required tools

- Allan key 4mm

- Check for any damage. _____
- Fold up foot plates (should be stiff and stay upright). (They should remain in the folded position, without falling back automatically) _____
- Are the foot plates evenly fitted and slanting upwards a little towards the middle? _____



Figure 4 Foot support

Date/Time PM Completed _____

Tango XL Serial Number: _____

Secure Bolt Replacement Due Date _____

Tech Signature _____