



Periodical preventative
maintenance

Tango XXL Shower-toiletchair

Tango XXL Shower-Toiletchair

51005800 (Standard)



Important!

The Tango XXL should be inspected at least once a year.

Repairs and maintenance should be undertaken only by staff authorised to do so by Lopital Nederland BV, and with original spare parts.

Making changes to the construction could affect the safety of the Tango XXL. This will also lead to Lopital's liability and warranty conditions being invalidated. As a result, the Tango XXL will no longer comply with the guidelines for medical devices.

Each inspection or repair must be entered to the log.

Manufacturer:

Lopital Nederland B.V.

Laarakkerweg 9

5061 JR Nederland

T (0031) 13 52 39 300

F (0031) 13 52 39 301

www.lopital.com

E-mail algemeen: info@lopital.nl

E-mail service: service@lopital.nl

Index

- 1. General condition of the chair 4
- 2. Wheels 5
- 3. Arm rests 6
- 4. Foot support 6

- 1. General condition of the chair
- 2. Wheels
- 3. Armrest/front closure
- 4. foot support

C= Check
 S= Service/lubricate
 A= Replacement advised
 V= Replacement



Month	12	24	36	48	60	72	84	96	108	120
General condition	C	C	C	C	C	C	C	C	C	C
PUR	C	C	C	C	A	C	C	C	C	A
Wheels	C	C	C	C	C	C	C	C	C	C
Armrest	C	V	C	V	C	V	C	V	C	V
Footrest	C	C	C	C	A	C	C	C	C	A

1. General condition of the chair

Month	12	24	36	48	60	72	84	96	108	120
	C	C	C	C	A	C	C	C	C	A

Required tools

- Touch- up pen Sikkens A77A2

- Visually check if the log, the instructioncart and the user manual are available. When it's not available, report this by the user.
- Visually check if all stickers are present and legible. Can you read the serial number yet?
- The CE Sticker should be present and clearly legible. This also applies to the sticker with inspection date.
- Visually check all painted parts for damage and corrosion. Touch up if necessary.
- Check PUR parts for severe discolouration (Brown in particular) and damage.
- Are all the screws present and securely fitted? Also that of the guide?

Present:

- Foot supports (not a failure item)

For reasons of hygiene, it is strongly recommended to replace the seat and back once every 5 years.

2. Wheels

Month	12	24	36	48	60	72	84	96	108	120
	C	C	C	C	C	C	C	C	C	C

Required tools

- Allen key 8mm



Figure 1 Wheel

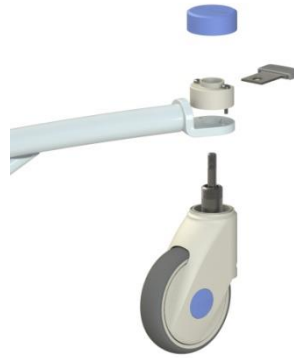


Figure 2 wheel exploded

- Check the attachment of the wheel (Tolerance)
- Check freewheel function + brakes.
- Do all four the wheels touch the floor on a flat surface? If not, check the chair right course.
- Brake lip intact?
- Check the tread.

Replace the wheels if they are damaged or worn.

3. Arm rests

Month	12	24	36	48	60	72	84	96	108	120
	C	V	C	V	C	V	C	V	C	V

Required tools

- Allen key 6mm

- Inspect its functioning, and check for any damage, corrosion.
- Check its room for motion, measured at the front point (replace if it is more than 4 cm.)
- Always treat corrosion.
- Are the arm supports evenly positioned?
- Is the secure bolt present?

Replace bolts once every 2 years.

Look for an exploded view on www.lopital.nl



Figure 3 Armrest

4. Foot support

Month	12	24	36	48	60	72	84	96	108	120
	C	C	C	C	C	C	C	C	C	C

Required tools

- Allan key 4mm

- Check for any damage.
- Fold up foot plates (should be stiff and stay upright). (They should remain in the folded position, without falling back automatically)
- Are the foot plates evenly fitted and slanting upwards a little towards the middle?



Figure 4 Foot support



Creating balance in healthcare



For more information, visit our website: www.lopital.com

Lopital Nederland B.V. Laarakkerweg 9, 5061 JR Oisterwijk, Postbus 56, 5060 AB Oisterwijk Tel +31 (0)13 5239300, Fax +31 (0)13 5239301, E-mail info@lopital.nl, www.lopital.nl

